AKITA® JET Year Pack

de: Gebrauchsanweisung
en: Instructions for use
fr: Mode d'emploi
it: Istruzioni per l'uso
nl: Gebruiksaanwijzing
da: Brugsvejledning
es: Manual de uso
pt: Instruções de utilização

Vectura Group plc
One Prospect West - Chippenham
Wiltshire, SN14 6FH – United Kingdom
Tel.: +49-89-897969-69 (INT). Tel.: 0800-832 88 72 (GER)
E-Mail: info@akita-jet.com
Item Code: 01DK0141 (DAW-00047 rev. 2)

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English: Instructions for use

WARNING: AKITA® JET Year Pack is an accessory to the AKITA® JET Inhalation System. Before operating this device read the entire AKITA® JET Inhalation System Instructions for Use. Correct and safe operation can only be achieved when the AKITA® JET Inhalation System Instructions for Use are followed.

1. General Warning and Caution

Accurate knowledge of and compliance with these Instructions for Use is required before using AKITA® JET Inhalation System. The user is liable, in all instances, when the device is operated incorrectly. All repairs must only be conducted by Vectura or by a service depot expressly authorized by Vectura. Vectura is not liable for any damage or malfunction resulting from improper use or from a use for which the device is not intended or indicated.

WARNING:

- Only use the device when cleaned and disinfected according to the AKITA® JET Inhalation System Instructions for Use. At multi patient use, the nebuliser must be sterilized as described in the Instructions for Use of the AKITA® JET Inhalation System.
- AKITA® JET nebuliser must be cleaned after each treatment and disinfected once a day.
- To ensure proper hygiene and performance, the AKITA® JET Year Pack may only be used for one year. Do not use any AKITA® JET part, component or device if damage is suspected.

2. Package Contents

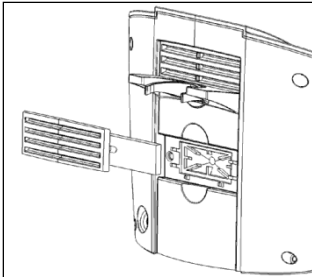
- Two AKITA® JET Nebuliser Sets, each set consisting of
 - One nebuliser (nozzle attachment: orange)
 - One nebuliser tube
 - One help flow tube
 - One help flow adapter
 - Tubing clips
- One Filter

3. Device Instructions

The AKITA® JET Year Pack can only be used in combination with the AKITA® JET Inhalation System. For important information regarding intended use and general instructions, refer to the Instructions for Use of your AKITA® JET Inhalation System.

3.1. Filter

Replace the dust filter annually. Leave the filter in the device between applications.

Remove the filter cover on the back of the control unit and remove the old filter. Insert the new filter in the filter connection and attach the filter cover.

3.2. Nebuliser sets

Replace both nebuliser sets annually. For information regarding assembly, use and care of your nebuliser set, refer to the Instructions for Use supplied with your AKITA® JET Inhalation System.

4. Disposal

The content of this package may be disposed as normal household waste.

5. Explanation of symbols used

Refer to Instructions for Use

Manufacturer of AKITA® JET Nebuliser Set and Filter:

Vectura Group plc

This device complies with the requirements of the Medical Devices Directive (93/42/EEC).

European Authorized Representative

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