

## Instructions for Use

---

700624



### **Warning:**

The FOX MOBILE Inhalation Set is an accessory to the FOX MOBILE Inhalation System. Read the entire instructions for use of the FOX Mobile Inhalation System before operating the device. Correct and safe operation can only be achieved if instructions for use of the FOX MOBILE Inhalation System are followed.

Please use the following email address if you have technical problems or questions concerning the FOX MOBILE:

Email: [fox.mobile@vectura.com](mailto:fox.mobile@vectura.com)



**Vectura Group Ltd**  
One Prospect West  
Chippenham, Wiltshire, SN14 6FH  
United Kingdom  
[www.vectura.com](http://www.vectura.com)

**Rx only**

## 1. Safety Instructions

Accurate knowledge of and compliance with these operating instructions is required before the FOX MOBILE is used. The operator of the FOX MOBILE is liable for its safe functioning in all cases when it is handled incorrectly. It must be repaired only by Vectura Group Ltd or by a service depot expressly authorized by Vectura Group Ltd. Vectura Group Ltd is not liable for any damage or malfunction resulting from incorrect use or from use for which the device is not intended.



### **Warning:**

- The FOX MOBILE is designed for single patient use. Do not share your FOX MOBILE with other people.
- Devices must be cleaned and disinfected according to the Instructions for Use of the FOX MOBILE Inhalation System.
- The Inhalation Set may only be used for a maximum of 14 days after 1<sup>st</sup> use, or 42 treatments, whichever comes first.
- Do not use the device if it is damaged.

## Instructions for Use

---

### 2. Package Content

- 1 x Nebulizer Reservoir
- 1 x Nebulizer Cap
- 1 x Mouthpiece
- 1 x Instructions for use (this document)

### 3. Intended Use, General Instruction, Use of the device

The FOX MOBILE Inhalation Set is an accessory to the FOX MOBILE Inhalation System. The FOX MOBILE Inhalation Set can only be used in combination with the FOX MOBILE Inhalation System. For information regarding Intended Use, General Instructions and the Use of the device refer to instructions for use of your FOX MOBILE Inhalation System.








### 4. Storage and Transport

No specific storage and transport conditions are defined for the FOX MOBILE Inhalation Set.

### 5. Service Life

Replace the Mouthpiece, Nebulizer Reservoir and Nebulizer Cap at the end of service life. The components delivered with the FOX MOBILE Inhalation Set may be used a maximum of 14 days after 1<sup>st</sup> use, or 42 treatments, whichever comes first.

### 6. Explanation of Symbols used

SYMBOL	INDICATION
	Refer to instructions For Use
	Manufacturer
	Reference number (article number)
	Serial number
	Medical device
	Unique device identifier
	Use-by date

Information as of August 2022

### 7. Disposal

Used and cleaned parts of the Inhalation Set can be disposed of in the normal household waste.