

Explanation of Symbols


Symbol Indication


 This device complies with the requirements of the Medical Devices Directive (93/42/EEC).

 Manufacturer

 Follow Instructions for Use!


 Use By







 Serial number

 Medical Device

 Unique Device Identifier

 Importer

 Indicates the packaging material is Non-Corrugated Fiberboard (paperboard)

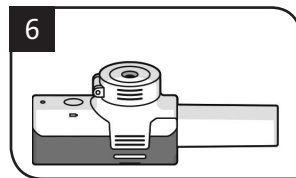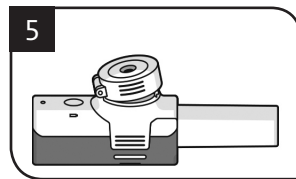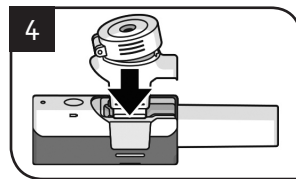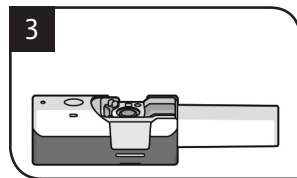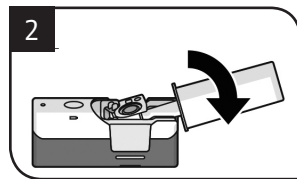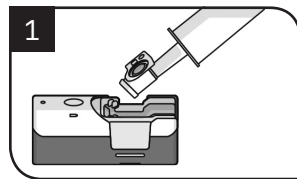

88282191

Breelib™

Monthly Pack

Instructions for Use



Vectura Group Ltd
One Prospect West, Chippenham, Wiltshire, SN14 6FH,
United Kingdom,
www.vectura.com



 0123

Vectura Ireland Limited
Block 2, Newtown Business and Enterprise Centre
Newtownmountkennedy
Co. Wicklow, A63 XV26
Ireland



08CN0201
270422



Packaging Technology Berlin GIRA
client: GVDE
Reference: 08CN0201
name: L-F-Breelib Monthly Consumables
colors: Black / **CYAN** / **MAGENTA** / **YELLOW**
version: 17.06.2022/03

page 1
PZ: 2782A-2
material-no.: 88282191
code-no.: 270422
IFU date: 270422
country: GB/-

Restricted Document
dimension: 255 x 85 mm

1. Safety Information

Warning

- ▶ The Breelib™ Monthly Pack is an accessory to the Breelib™ Inhalation System. Read the entire Instructions for Use of the Breelib™ Inhalation System before using the device. Correct and safe operation can only be achieved if Instructions for Use of the Breelib™ Inhalation System are followed.

- ▶ Do not share your Breelib™ with other people as it can lead to infections. The Breelib™ is designed for single patient use.
- ▶ Ensure the Breelib™ has been properly cleaned and disinfected before use according to the Instructions for Use of the Breelib™ Inhalation System.
- ▶ Do not use the components of the Monthly Pack for more than one month to maintain constant performance.

- ▶ Do not use the Breelib™ if it is damaged or modified. Only use accessories supplied by an authorised service partner.
- ▶ Only Vectura Group Ltd or a service partner authorised by Vectura Group Ltd is permitted to repair the Breelib™. Vectura Group Ltd is not liable for any damage or malfunction as a result of unauthorised repair.

2. Package Content

- ▶ Instructions for Use (this document)
- ▶ Mouthpiece
- ▶ Nebuliser unit

3. Intended Use

The Breelib™ Monthly Pack is an accessory to the Breelib™ Inhalation System. The Breelib™ Monthly Pack can only be used in combination with the Breelib™ Inhalation System. For information regarding intended use, general instructions and the description of the device refer to the Instructions for

Use of your Breelib™ Inhalation System.

4. Storage and Transportation

No specific storage and transport conditions are defined for the Breelib™ Monthly Pack.

5. Service life

The components of the Breelib™ Monthly Pack are intended for repeated use. The service life of the Breelib™ Monthly Pack is limited to one month.

6. Disposal

Components of the Monthly Pack may be disposed of with normal household waste.

Bayer
882 82191